

IMA OTOMATİK ÇALIŞMA PROSEDÜRÜ VE GRAFİKLER

Assay type	2 Point End
Reaction time (min.)	10
First reading point	90 Second
Second reading point	600 Second
Wavelength (sub/main)	800/470 nm
Reaction direction	Increase
Units	ABSU
Sample pipetting volume	35 μ L
Reagent 1 pipetting volume	120 μ L
Reagent 2 pipetting volume	9 μ L
Calibration mode	Linear
Standard 1	0.0 ABSU
Standard 2	100.0 ABSU
Decimal	1

